Suction Cup Grab Bars

RETAIL BOX

INSET B

INSET C

INSET D

- Installs and removes without tools or professional installation
- Can be installed at any angle
- Large suction cups provide an extremely strong hold; suction cups are 3.75" in diameter
- Release levers make installing and removing the suction cup grab bars quick and easy
- Red/Green color indicators on top of the handle shows "safe secure" suction or to "re-affix" suction
- Avoid grout lines for proper placement
- Will not damage property
- Use to assist with balance only
- Do not use to leverage body weight
- Always check to make sure the bar is securely attached before using
- Uninstall and reinstall as needed
- Retail Packaged

PRODUCTS

Description

RTL13083	White & Beige 12" Suction Cup Grab Bar, 3/cs
RTL13085	Black & Chrome 12" Suction Cup Grab Bar, 3/cs

SPECIFICATIONS

Description

Length	12"
Weight	.5 lbs.
Carton Shipping Weight	3.5 lbs.
Warranty	Limited Lifetime

driveTM

www.drivemedical.com

Rev.1.04.29.16

Suction Cup Grab Bar

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Item # RTL13083, RTL13085

EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

driveTM

grab bar

RTL13083

RTL13085

Easily and securely attached without drilling.
Provides security and the convenience of mobility.
Won't rust or corrode.

Drive Medical Suction Grab Bars are convenient, functional, and offered in a variety of sizes to fit any lifestyle. They can be attached quickly and are just as easily removed! The suction pads are specifically designed to adhere to any smooth, non-porous surface without drilling! Push down on the vacuum toggle lever and the handles will remain securely in place.

CAUTION

Grab bar is to be used as a balance assist only. Not for supporting body weight. Always make sure the grab bar is securely attached before each use and always reattach if loose.

Always test the holding grip on each surface each time you apply the suction pad to prevent accidents.


Rev.1.04.29.16

installation instructions

1. Check the vacuum suction discs and the surface it is to be attached to.
2. Clean the surface thoroughly.
3. Press the rubber discs firmly to the surface. The tilting vacuum levers must be loose at this point and the whole of each rubber disc must lie on the surface.
4. Ensure that the rubber discs are completely on the tile area or other surfaces so that the discs do not cover any grouting lines or other joints. The discs should be positioned at least 1/2" away from all grout lines or joints.

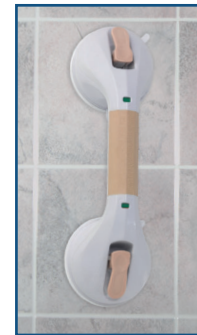INCORRECT

CORRECT

5. First, press down one tilting lever and then press down the second. The resistance of the suction pressure generated must be clearly felt. The greater the resistance felt on the vacuum lever, the better the suction power of the grab bar. The indicator on the top of the suction cup will change from RED to GREEN when a safe and secure hold is achieved. If you use the bar regularly, then you will get a feel for the vacuum strength and the suction power generated.

SECURE

REAFFIX

6. Before each use, test the firm hold of the suction device making sure both indicators are GREEN. If the adhesion seems insufficient, then release the rail and reattach again. If the required suction power has not been achieved, the rail should not be used.

removal instructions

1. Release both latches.
2. Slide fingertip under rubber seal at each end to release suction.

maintenance

1. Always keep the rubber suction discs clear of oil and dust.
2. Clean the rubber discs with standard household cleaning agents.
3. Never expose the rubber discs to sunlight for prolonged periods of time otherwise they will quickly lose their elasticity.
4. Check the rubber discs for possible deterioration. If the suction cups show damage, then replace immediately.

lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2016 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2016 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.